


Getting Started with OrthoLogic Total Diagnostics and Treatment Design

Technical Support: 706-474-3353

In order to use the *OrthoLogic Total Diagnostics and Treatment Design*, you will be asked to go to the case management portal:

<https://orthologic.facialbeautyinstitute.org>

If you do not have an account, please click on the Register link below the login button.

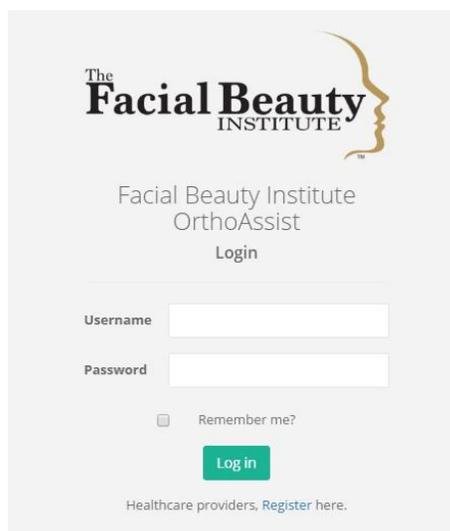

The screenshot shows the login page for the Facial Beauty Institute OrthoAssist portal. At the top, the logo for 'The Facial Beauty INSTITUTE' is displayed, featuring a stylized profile of a face. Below the logo, the text reads 'Facial Beauty Institute OrthoAssist Login'. There are two input fields: 'Username' and 'Password'. Below these fields is a checkbox labeled 'Remember me?'. A green 'Log in' button is positioned below the checkbox. At the bottom of the form, there is a link that says 'Healthcare providers, Register here.'

After you click on the Register link, you will be prompted to supply details for your portal login.

- Username
- Email Address
- Password

Your password must contain upper and lower case letters, and numbers.

After you have successfully created your login account, you will be asked to enter your provider information. You will be asked to provide the following information:

- State License (or Country License)
- First and Last Name
- Middle Name or Initial
- Gender
- Degree

- Contact Name
- Email Address
- Office and Mobile phone numbers
- Dental Practice Name
- Address 1 and Address 2
- City
- State or Province
- Country
- Zip or Postal Code
- Contact Information (You or an assistant that enters the case information)
 - Name
 - Email
 - Telephone

All fields, except Middle Name or Initial and Address 2, are required.

After you have completed the required information, click the Save button located at the top right of the screen.

Your provider information will be saved, and you will be directed to the provider Account Information screen. Here you will be able to review and edit changes to your account information, add and manage payment credit or debit cards, and change your password.

You will be required to provide payment information before completing treatment plans and diagnostic cases. Click on the Add Card button to enter a card.

Add a Credit or Debit Card ✕

CARD TYPE

CARD NUMBER

EXP MONTH **EXP YEAR** **CVC CODE**

NAME ON CARD

Address

City **State** **Zipcode**

Country

Make this card the default payment card.

You will be prompted to enter your credit or debit card information. Click the Save button to save your credit card. This will place a \$1 USD authorization hold on your card to verify that it is valid. This hold will be removed in a couple of days. This authorized amount will not be deducted from your account.

Your payment card information will be saved in a secure PCI level-1 compliant facility. Only your card number and card's billing information will be saved. The card CVV code will be discarded after verifying the card.

Charges from the *OrthoLogic Total Diagnostics and Treatment Design* portal will appear as FACIALBEAUTYINST on your monthly charge statements.

If you should ever receive a call from your bank or card provider institution inquiring about charges from the portal, these charges may appear as from either Stripe, or online payment at Stripe.com. If you should ever have any questions about charges to your card that are not clearly identified as FACIALBEAUTYINST that you feel may be charges for treatment plans or diagnostic services, please contact Facial Beauty Institute support, support@facialbeautyinstitute.com, for assistance.

OrthoLogic Total Diagnostics and Treatment Design

There are four requirements that must be completed in order to submit your completed case to OrthoLogic Total Diagnostics. The quality of the OrthoLogic diagnostic analysis is dependent on the quality of the data that you provide for your patients. Tracing of the patient data is a science that is highly dependent on the images provided. All JPG files for 2D images should be high quality files.

You may submit either 2D information or 3D information. CBCT scans are typically lower radiation than x-rays. We recommend that you submit a 3D scan. Every facial structure is different. It is important to have scans that provide all of the key landmarks for analysis. CBCT scans should cover from 20mm above the top of the eyebrow to 20mm below the Pogonion.

1. Patient information - name, birth date, dentition, chief complaint, etc. You are prompted for these when you create a new case, and what you enter here is also editable later. Data validation that occurs when you save this information will prevent you from omitting any required data. All fields are required information, except that if you specify a specific age, you do not have to specify the birthdate. If you select Other as Ethnic Group, the other ethnic group text box becomes enabled, and is required. NOTE: The case name must match the name on a CBCT file that is submitted. We never 'see' your patient and the reviewing doctors will reject a case if the names do not match.
2. Patient pictures - you need to upload 8 pictures to create a standard case picture set. The required pictures are frontal repose, frontal smile, right profile (left-facing profile, so the patient's right side should be pictured), maxillary occlusal, mandibular occlusal, occlusion right, frontal occlusion, and occlusion left. These are very important to help identify potential issues with your patients.
3. Case survey - this is a series of 27 radio buttons that describe attributes of the case. All of these must be specified in order to successfully save this data. If you miss any, the overlooked responses will be highlighted in a red outline.
4. Imaging file uploads - the required imaging files can be either 2D ceph and pano, or 3D CBCT scans. If you upload 2D images to a case, we require both the ceph and pano. Uploaded 3D imaging must have a field of view that includes the nasion at the top, to the bottom of the chin. Sometimes this may be difficult if the patient has a long face or the machine itself has a limited field of view. In those cases, it is possible to take two scans that overlap in the middle, and the software can stitch them together into a single image for tracing. The 3D field of view must also extend to the tip of the nose, so the patient must be placed far enough back so that the scan will include up to the tip of the nose.

To create a new Total Diagnostics and Treatment Design case, click on New Case under Diagnostics in the left navigation menu.

All information fields are required. When you enter a birth date, the age will be automatically calculated for you. Scan date is the creation date of any images, 3D DICOM, or 2D Panorex or OPG files you attach to your case. Treatment plan date is any date you choose as a point of reference for the case timeframe.

Click the Save button at the top to save your case information. After successfully saving your new case, you will be directed to a Total Diagnostics screen where you will complete the remaining three requirements. The patient information you previously entered is at the top of this page. To make any edits to this information, click on the gray pencil icon located in the upper right corner of this section.

Below the patient information, you will see the Case Requirements Progress section. As you complete each requirement, a green checkmark will appear beside it. After all four requirements are completed; the Submit Case button will become enabled.

Patient Pictures

In this section, you will upload the 8 patient pictures and create a formally arranged picture set. Click the blue Upload Pictures button (located below the 8 picture boxes) to open the picture file upload area. A box with a dashed outline will appear. You can either drag and drop pictures from a file explorer window into this area, or click on the upload area to open a file explore window to locate the pictures you wish to upload.




When you click on the Upload pictures button, the upload area will appear.



After you select your pictures, they will be queued for upload. Click the blue “Remove Picture” below each picture to remove any picture from the upload queue. Click the green Upload button to start uploading pictures to the server. A progress bar will appear over each picture to indicate the upload progress of each picture. It is required to include these pictures. Many times the radiologist will look at the patient photo to determine head alignment or will need a profile reference if the CBCT scan cuts off part of the patient’s nose.

A green File Upload box will appear in the upper right corner of the screen after the pictures have finished uploading, and the uploaded pictures will be shown along the left side of the 8 picture placeholder boxes. Place each picture by dragging it into the appropriate picture set placeholder. If a picture is incorrectly placed into a placeholder box, hover over the incorrect placement and a small ‘x’ will appear. Click on the ‘x’ to remove the photo from that placeholder box. You can then move the picture to its correct placeholder box.

Here is an example of good patient photos:

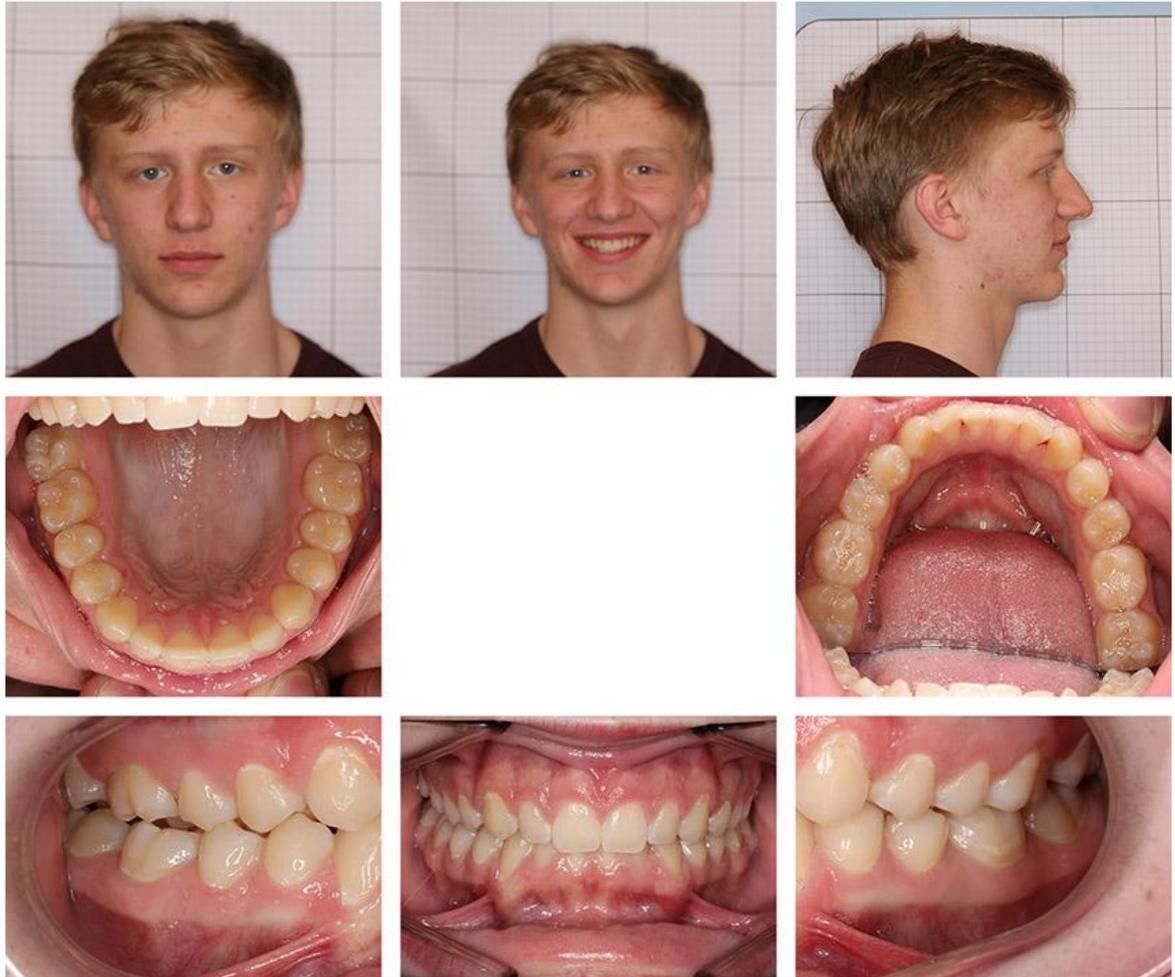

Case Survey

Scroll down below the patient pictures to find the Case Survey. Here you will provide specific case details by using a series of radio button options. A response is required for each of the 27 questions. Each question has an Unknown option for case details which you do not know yet, or which will be determined during the Total Diagnostics and Treatment Design process.

1. Mx Position = Normal, Retrusive, or Protrusive
2. Treatment Phase = Select continuous for adult patients; Select 2 phase for younger patients that will require a removable appliance; NOTE: TMJ splint therapy is NOT a two phase process
3. Dental Class = The left and right sides can have different class and division; Note: This is the canine classification and not the molar; Division should be the same for both sides
4. Dental Bite = Normal, Deep, Open

5. Skeletal Class = based on profile, I, II, or III
6. Skeletal Growth Pattern = Neutral, Horizontal, Vertical (based on mandibular plane angle)
7. MN Size = Neutral, Retrognathic, Prognathic
8. MX Arch Width at 1st molars = Neutral, Constricted, Over Expanded (if any part of the arch appears constricted, choose constricted)
9. MN Arch Width = Neutral, Constricted, Over Expanded
10. MX Midline = Neutral, Left, Right
11. MN Midline = Neutral, Left, Right
12. MX Palatal Depth = Normal or High (over 14mm), Shallow (under 14mm)
13. Crossbite = None, Posterior, Anterior, Both
14. MX Crowding = Spacing, None, Slight, Moderate, Severe
15. MN Crowding = Spacing, None, Slight, Moderate, Severe
16. MX 1st Molar Rotation =
 1. Left – None, 10% or less, More than 10%
 2. Right – None, 10% or less, More than 10%
17. MN 1st Molar Rotation =
 1. Left – None, 10% or less, More than 10%
 2. Right – None, 10% or less, More than 10%
18. Curve of Spee =
 1. Left – Normal, Slight, Moderate, Deep, Reversed
 2. Right – Normal, Slight, Moderate, Deep, Reversed
19. MX Cant = None, Anterior, Posterior, Both
20. MN Cant = None, Anterior, Posterior, Both
21. Asymmetry of Face = None, Slight, Moderate, Severe
22. Head Posture = Normal, Forward, Rotated, Both Forward and Rotated
23. Swallow Pattern = Normal, Anterior Thrust, Lateral Thrust, Scalloping, Combination
24. Ankyloglossia = None, Over 50%
25. Lip Seal While Breathing = Yes, No, Not Sure
26. Nasal Breather = Yes, No, Not Sure
27. Patient has TMJ Clinical Symptoms = No, Yes

Click the green Save Case Survey button below the Case Survey option selections. After your selections are saved, a green checkmark will temporarily appear next to the Save Case Survey button.

File Upload

Click the blue Upload Files button in the bottom right corner of the screen. A box with a dashed outline will appear. Drag files from a Windows Explorer window, or click in the file upload area to open a file explore window to navigate to the case files you wish to upload.

File upload

Diagnostics / File Upload

Case File Upload - Curley Legrew

Age: 34	Chief Complaint: crooked teeth
Birthdate: 7/18/1982	Ethnic Group: Caucasian
Dentition: Permanent	Sex: M

File Uploads


File Name	Size	Type
<div style="background-color: #00a651; color: white; padding: 2px; display: inline-block;">Return to case edit</div>		

File Upload

Click, or drop files, here to upload

The files you select will be queued for upload and will appear as thumbnails in the file upload area. When you click the green Upload button above the file upload area, you will be prompted to identify the type of each file you are uploading, and optionally enter a brief description. Click the green Upload button in the File Upload Types window to begin the actual upload. The amount of upload time will depend on the size of your files and the speed of your internet connection.

The type of files you can upload to your case can be any typical image format such as jpeg, gif, or png. To decrease file upload times and minimize storage requirements, we recommend that you avoid uploading uncompressed file formats such as tiff and bmp. Other typical upload file formats are 3D DICOM and 2D Panorex or OrthoPantomograph (OPG). The approximate maximum upload file size the server will accept is 900MB.

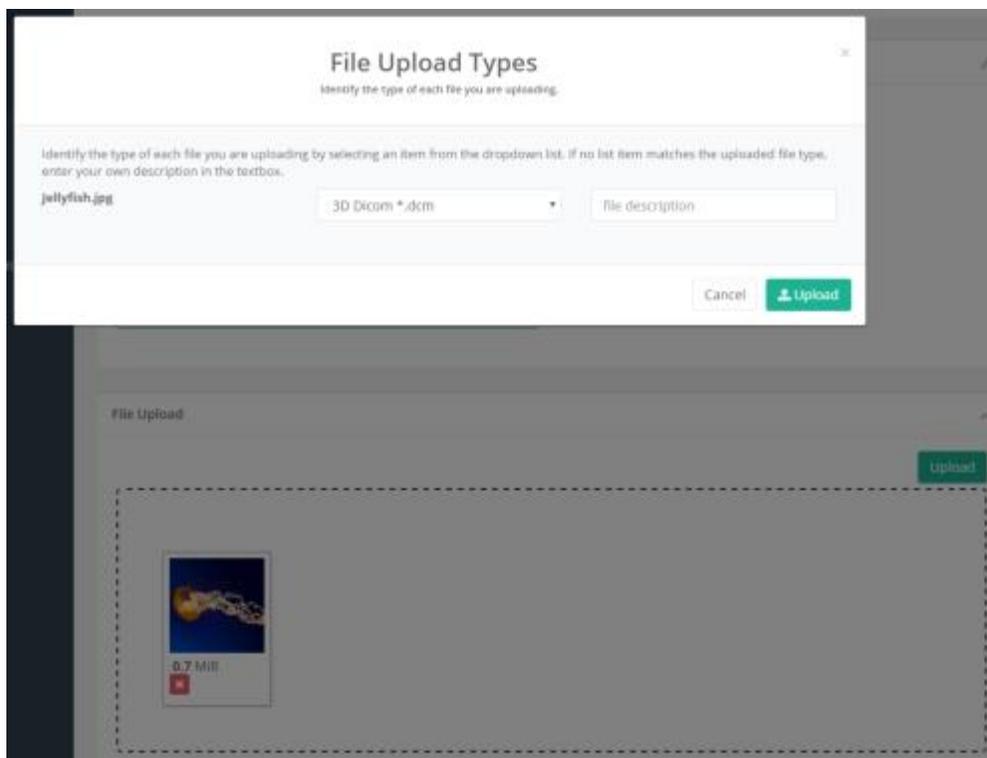

The location of the 3D DICOM files on your server can vary depending on 3D equipment and imaging software vendors. Typically, DICOM images are stored in a folder as a series of hundreds of small, separate files, which are typically about 250 kilobytes in size. Once you have located this folder, you must zip it to be able to upload them altogether to OrthoLogic as one zipped archive.

Most imaging systems support some means to export DICOM imaging. It is also common that these systems provide some means to export imaging to a bundled image viewer and data executable program. This is sometimes referred to as “Pack and Go”, or “Wrap and Go”. The imaging format contained in bundled viewer exports is not usable. We require actual DICOM files. Some imaging systems give you a choice of exporting to multiple DICOM files (one file per slice) or to a single DICOM file. Either way is acceptable, but in either case, please zip the image data before uploading to OrthoLogic. For assistance with exporting DICOM from any specific imaging system, please contact Facial Beauty Total Diagnostics support to schedule a support call to walk you through the process.

NOTE: Please enter the patient name on the DICOM scan. We check the name to ensure that the name in the DICOM file is the same patient name as the files submitted. It is easy to make mistakes in uploading the wrong patient DICOM file with your submission.

The easiest way to zip files or folders on a Windows computer is to either right-click on a folder, or on a collection of selected files, and choose “Send to compressed folder” on the context menu. This will create a new zip file that contains all the selected files or folders. This new zip file will be named using the first file or folder in your selection. After the zip

file is created, you may rename it with a filename that contains the patient name or identifier.

There are two types of scan data that you can provide to OrthoLogic. One is a 3D CBCT file. This file must include image data from 20mm above the upper eyebrows to 20mm below the Pogonion. The second type of data you can provide is two x-rays: a good quality ceph and a pano (both are required). These files should be zipped prior to uploading.


If you have any difficulty or need assistance locating DICOM files, please contact Facial Beauty Institute support, support@facialbeautyinstitute.org. Please include with your request any information you have about your equipment, such as manufacturer and model or series information, and name and version of your imaging software. Please contact support with your questions or need of assistance with any features of OrthoLogic Total Diagnostics and Treatment Design.

Submitting Completed Cases

After these 4 requirements are satisfied, the Submit button will become enabled for you to submit your case to us to begin working on. The single most common holdup to case progress is the imaging. If the scan we receive does not have the required landmarks, it is not possible to do the tracing and we will have to ask for a new scan. The field of view is the most common imaging problem. Occasionally we receive scans in which the patient has moved during the scan, or have other image quality problems.

This summarizes all our requirements for submitted cases. The 3D imaging is the most common source of problems that can delay case processing. Please take care to properly position the patient in the machine so that the scan contains the full required field of view with good image quality.

An example of the CBCT requirements is presented in the following picture:

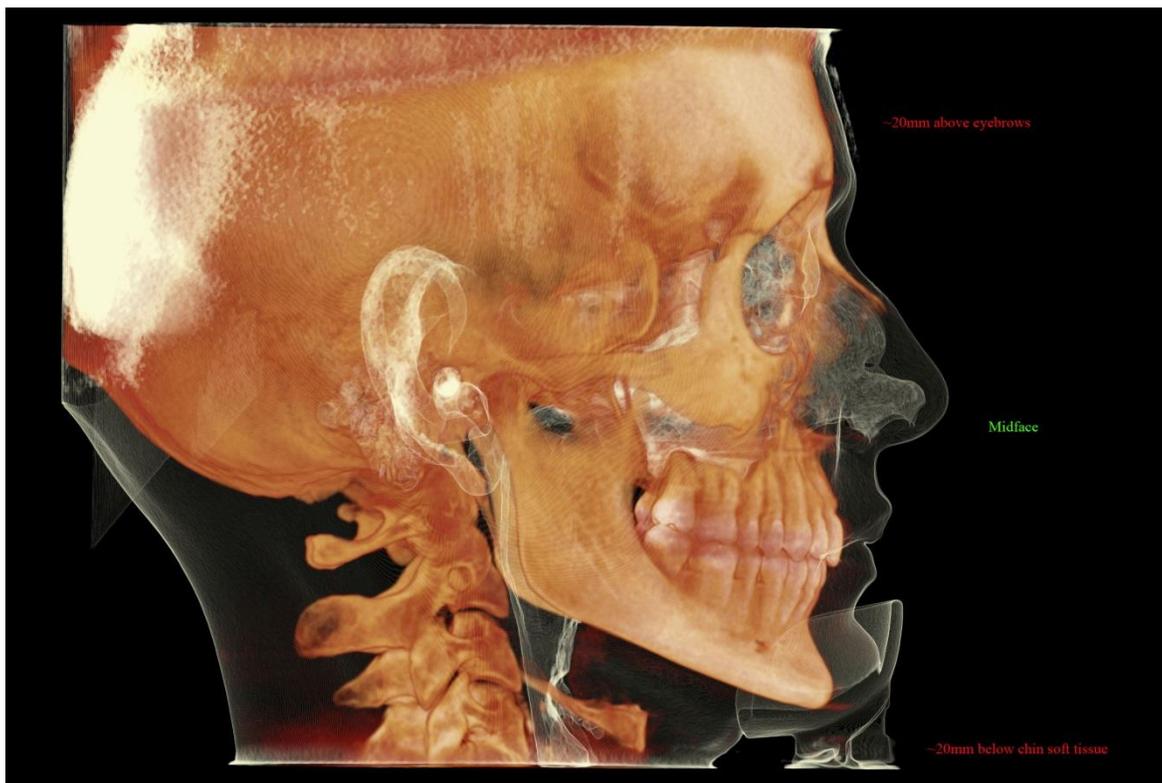

Carrie McGillicuddy

Age: 7	Chief Complaint: sad smile
Birthdate: 6/17/2009	Ethnic Group: Caucasian
Dentition: Mixed	Sex: F
Create Date: 2/21/2016	Submit Date:
Treatment Plan Date: 1/15/2016	Image Scan Date: 9/12/2016
Missing Teeth:	Impacted Teeth:

Comments

type your comments here...

Save

Case Status: **New**

Case Requirements Progress

General case information	✓
Case survey	✓
File upload	✓
Patient pictures	✓

After all case requirements are completed, click Submit Case to place your case in the work queue, and notify the Facial Beauty Institute.

Submit Case

By clicking on the green Submit Case button, you are notifying OrthoLogic Total Diagnostics that your case is ready to work on, and it will then appear in the case work queue.

You will be able to monitor the status of your case by clicking on Cases under Diagnostics in the left navigation menu. The Case List view contains a table that lists all your cases. The patient column contains hyper-linked patient names. Click on the patient name to go to a detail view of that case to check on case status and any comments or questions from the reviewer who is working on your case.

Case Status: **Completed**

Case Requirements Progress

- General case information ✓
- Case survey ✓
- File upload ✓
- Patient pictures ✓

Completed Case Files

Completed Files Download



<input type="checkbox"/>	File Name	Size	Description
<input type="checkbox"/>	 Chrysanthemum.jpg	879394	nice flower
<input type="checkbox"/>	 Hydrangeas.jpg	595284	3D Dicom *.dcm
<input type="checkbox"/>	 facial assessment Emma James id(21).docx	12634	facial assessment report
<input type="checkbox"/>	 Orthodontic Consent Form-002.docx	17774	
<input type="checkbox"/>	 Orthodontic Consent Form-001.docx	18841	
<input type="checkbox"/>	 MN Osseo-restoration Appliance Instructions.docx	100205	
<input type="checkbox"/>	 MX Tongue Thrust Appliance_without lateral loops.docx	116267	
<input type="checkbox"/>	 Desert.jpg	845941	so dry

Completed cases will be moved into Completed Cases under Diagnostics. When this happens, you will receive an email notification, and you can review the completed case documents by clicking on the Completed Cases menu under Diagnostics in the left navigation menu. The completed cases view will display a table showing all completed cases. Click on the hyper-linked patient name to review the details for that case. Here, you will find a table that shows all the completed work output from the case such as treatment plan, diagnosis, tracings, radiologist report, and appliance instructions. You can individually download each report or file in the table, or select all or some of the files to download as a single zip archive to your computer.

If you have any questions, please contact:

Technical Support via Google Voice: 901-316-7991